Scan for more information

Check out our new learning platform Reliance U! www.RelianceU.tech

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com


EMERGO EUROPE


Prinssgracht 20, 2514 AP The Hague, The Netherlands

AUSTRALIAN SPONSOR EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

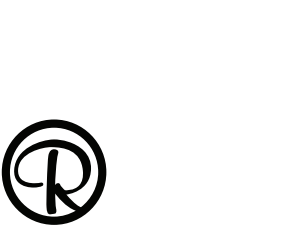PORC-ETCH™

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INSTRUCTIONS FOR USE

RELIANCE®
ORTHODONTIC
PRODUCTS

RelianceOrthodontics.com

ENGLISH TRANSLATION

INTENDED USE: Reliance Porc-Etch™ with Barrier Gel is intended for use in the preparation of glazed dental porcelain bonding. The Porc-Etch™ will prepare porcelain surfaces for the application of Porcelain Conditioner.

INSTRUCTIONS FOR USE:

- Use caution – Porc-Etch™ contains 4% hydrofluoric acid. Placement of a rubber dam is required. Do not allow material to contact gingiva or soft tissues.
- Prophy and sandblast the surface to be bonded. Rinse and Dry.
- Thoroughly isolate and place Reliance Barrier Gel on the gingiva. Carefully apply a thin coat of Porc-Etch™ to the crown. Wait 4 minutes and wipe Porc-Etch™ off with a cotton roll or gauze pad. Rinse thoroughly with water while using a high speed suction.
- Dry surface thoroughly and apply one THIN coat of Reliance Porcelain Conditioner. Wait 2 minutes. Thoroughly air dry.
- Apply one coat of Assure® Plus to the conditioned porcelain surface. Lightly air dry and light cure. Place Adhesive and bracket.

⚠️ **CAUTION:** Hydrofluoric acid is a very powerful and aggressive chemical. It is a severe eye and tissue irritant. If accidentally splashed into the eye, flush with copious amounts of water for 15 minutes and seek immediate medical attention. Hydrofluoric acid also releases a vapor that irritates respiratory passageways. Chronic inhalation of fumes is dangerous and can cause damage. Extreme care must be taken to protect the patient, operator and assistant. These individuals must wear protective eyewear. The operator and assistant must wear protective gloves. Rubber dam isolation must be used for intraoral porcelain repair. If Porc-Etch™ comes in contact with other tissues: rinse affected area immediately with copious amounts of water for several minutes. Injury may result if etchant is allowed to remain on the skin or mucosa for any length of time.

⚠️ NOTE: A NEW APPLICATOR IS REQUIRED FOR EACH PATIENT

TEMPERATURE LIMITATION: Store at room temperature (15-30°C).

TO REORDER: Ref# POE – 7 g Porc-Etch with Barrier Gel Ref# GEL – 3 mL Barrier Gel only

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: Federal law restricts this device to sale by or on the order of a dental professional.

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